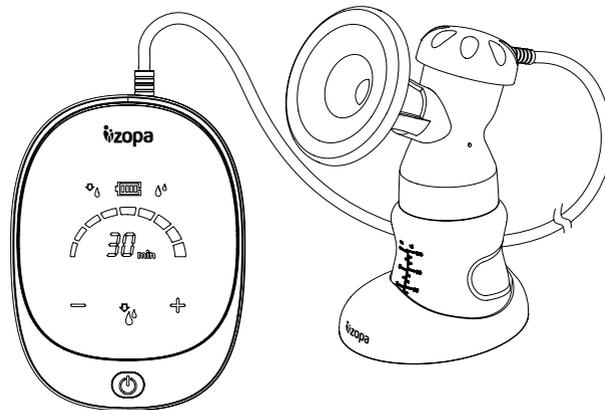
MANUÁL PRO ELEKTRICKOU ODSÁVAČKU MLÉKA **COMFY PRO** ZOP081105

- (SK) PRÍRUČKA PRE ELEKTRICKÚ ODSÁVAČKU MLIKA COMFY PRO
- (EN) MANUAL FOR ELECTRIC BREAST PUMP COMFY PRO
- (DE) GEBRAUCHSANWEISUNG FÜR ELEKTRISCHE MILCHPUMPE COMFY PRO
- (HU) ELEKTROMOS MELLSZÍVÓ HASZNÁLATI ÚTMUTATÓ COMFY PRO
- (PL) LAKTATOR ELEKTRYCZNY – INSTRUKCJA OBSŁUGI COMFY PRO
- (IT) MANUALE PER IL TIRALATTE ELETTRICO COMFY PRO
- (SLC) PRIROČNIK ELEKTRIČNEGA APARATA ZA SESANJE MLEKA COMFY PRO
- (FR) NOTICE D'UTILISATION TIRE-LAIT ELECTRIQUE COMFY PRO
- (ES) MANUAL PARA EL EXTRACTOR DE LECHE ELÉCTRICO COMFY PRO

<u>CZ</u>	<u>4</u>
<u>SK</u>	<u>9</u>
<u>EN</u>	<u>14</u>
<u>DE</u>	<u>19</u>
<u>HU</u>	<u>24</u>
<u>PL</u>	<u>29</u>
<u>IT</u>	<u>34</u>
<u>SLO</u>	<u>39</u>
<u>FR</u>	<u>44</u>
<u>ES</u>	<u>49</u>

MANUÁL K POUŽITÍ

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USER'S MANUAL

IMPORTANT! KEEP FOR FUTURE REFERENCE

MANUAL FOR ELECTRIC BREAST PUMP COMFY PRO

- Unique soft silicon mimics the baby's sucking action and allows effective milk expression
- Pleasing modern design, cutting edge high quality model
- Multiple power options for ease of use

WARNING

1. Only for use by one person. Use by more than one person may present a health risk.
2. After use, always unplug the adapter.
3. When cleaning, ensure the adapter has been unplugged.
4. Do not clean the cap with water. If you find water in the cap, remove the membrane base from it and leave it to dry out before using it again.
5. Check that the tube is clean and there are no water droplets within it.
6. No components should be sterilised in a microwave oven or steriliser.
7. Do not subject the pump and motor unit to high temperatures or flames and keep out of the reach of children.
8. Heat and massage will reduce any pain which may occur while expressing milk.
9. Product design changes reserved.

FUNCTION AND PROPERTIES

1. Product functions

- (1) Microprocessor controlled suction mimics your baby's discontinuous sucking for maximum effectiveness.
- (2) In stimulation mode, you can choose 9 levels of suction force, which massage your nipples and promote milk creation and expression. In suction mode, there are 9 levels of suction, which mimic your baby's sucking rhythm.
- (3) The memory function remembers your favourite suction level in line with your most recent setting.

2. Product properties

- (1) The membrane's unique design prevents milk backflow.
- (2) Soft silicon pads secure massage and comfortable expression.
- (3) Adapters for narrow-neck bottles included in pack.
- (4) The LED screen displays all the information needed about your current setting.
- (5) Option to use 4x AA batteries if a socket is unavailable.

TECHNICAL PARAMETERS

Power adapter

Power supply: 100V-240V ~ 50/60Hz

Output power: 6V 1A

Power 3.6W

Stimulation speed 100T/min

Suction speed 26-60T/min

Stimulation force: -0.002 ~ -0.018Mpa (max), 9 setting levels

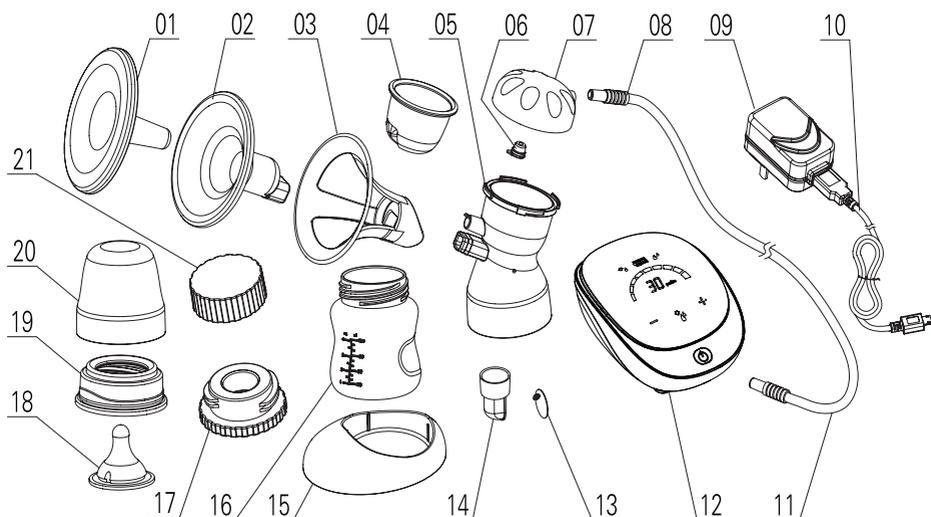
Suction force: -0.13 ~ -0.036Mpa (max), 9 setting levels

Timer: 30 minutes

Noise: ≤60dB(A)

Operating conditions: 5-40°C, RH≤90%

Electric shock protection: Class II AC Adapter, Group III motor.

PRODUCT DESCRIPTION

01.	Silicon breast pad guard	1 PP	12.	Motor unit	1
02.	Silicon breast pad	1 SIL	13.	Vent membrane	2 SIL
03.	Breast attachment	1 PP	14.	Vent	1PP
04.	Membrane	1 SIL	15.	Bottle stand	1 PP
05.	Pump body	1 PP	16.	120ml bottle with wide neck	1 PP
06.	Membrane base	1 SIL	17.	Adapter for bottle with narrow neck	1 PP
07.	Cap	1 PP	18.	Teat	1 SIL
08.	Tube connector	3 PP/TPE	19.	Securing collar	1 PP
09.	Power adapter	1	20.	Bottle cap	1 PP
10.	USB cable	1	21.	Bottle close cap	1 PP
11.	Tube	2 SIL			

ASSEMBLY AND CONNECTION

The following parts which come into contact with breast milk when used should always be cleaned and sterilised before every use: silicon breast pad, membrane, bottle attachment, vent membrane, vent, bottle, bottle close cap, adapter for bottle with narrow neck.

A. Breast pump assembly (breast attachment)

Fit the silicon breast pad (02) to the breast attachment (03) and affix to the pump body (05), then check that they are firmly attached.

B. Breast pump assembly (membrane)

Insert the membrane (04) into the pump body (05), affix the cap (07) and screw on firmly so no air can escape when you connect the tube (08).

C. Breast pump assembly (vent)

Attach the vent membrane (13) to the vent (14). The membrane (13) should stick to the vent (14) to prevent air from escaping. Now attach the vent (14) to the pump body (05). Check they are firmly attached so no air can escape.

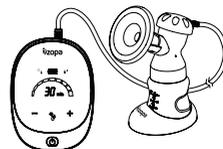
D. Breast pump assembly (bottle and bottle stand)

Screw the bottle (16) onto the pump body (05) and insert it into the bottle stand (15).

CONNECTING THE BREAST PUMP TO THE MOTOR UNIT


A. Motor unit and breast pump

Connect the pump using the connectors on its ends to the pump body cap (07) and to the motor unit (12). The connection should be firm to prevent air from escaping.


B. Adapter and USB cable

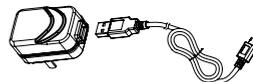Connect the USB cable (10) to the power adapter (09).

C. Motor unit and power adapter

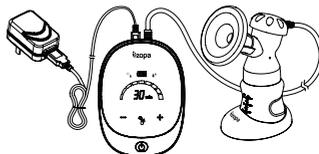
Insert the micro-USB connector into the motor unit as displayed on the diagram and insert the power adapter into the socket. The LED display should turn on then shortly turn off, showing it is on standby.


D. Using batteries

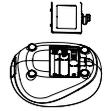
Picture 1: Open the battery cover.

Picture 2: Insert 4 new AA alkaline batteries in the correct position as shown. Place the batteries over the ribbon to ease battery removal.

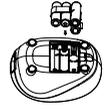Picture 3: Put the cover back in place and click on.

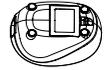
USE

1. Before every use

- (1) Check the pump is clean and does not contain any liquid of any sort.
- (2) Clean, press and massage your breast, nipple and areola; relax to avoid pain when extracting milk.

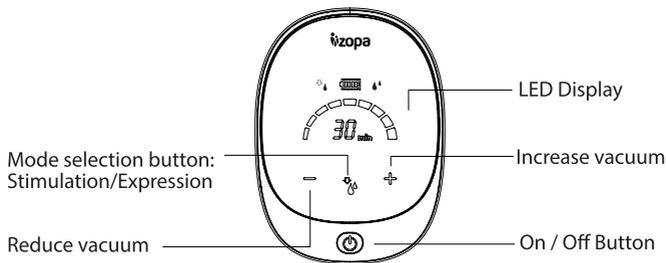2. Control panel description

3. Turning on/off

- (1) After plugging in the pump and pressing  the on button, the motor will run in stimulation mode at the lowest level for a period of 2 minutes, after which it will automatically move to suction mode. The timer is set for a total time of 30 minutes.
- (2) To turn off, press  the off button and the motor will switch to standby, with your most recent setting saved in the memory.

4. LED screen

- (1) After pressing , the Phase button, you can select between expression  and stimulation .
- (2) The LED screen displays the set suction force.
- (3) When using with batteries, the screen displays the current battery level ; if they need replacing the icon  flashes.

5. Setting suction force and speed

- (1) During stimulation, you can press the + and – buttons to set 9 suction speeds.
- (2) During extraction, you can press the + and – buttons to set 9 suction speeds.
- (3) If you turn the pump on after it has been unplugged or the batteries have expired, there are three preset suction force levels in extraction mode , and 1 level for stimulation  mode; these are factory settings.

6. Timer

- (1) When you plug the pump in and press  the on/off button to start suction, the timer will be set to 30 minutes, and once this time is up the motor unit and LED diodes will switch off.
- (2) If you press  the on/off button while pumping, the motor unit will stop and the timer will reset. If you press  the on/off button again, the motor unit will start again and the timer will be set for 30 minutes again.

7. Switching between stimulation/extraction modes

- (1) When you plug the pump in and turn it on , stimulation mode at 1 level of suction force begins; after two minutes it automatically switches to extraction mode with 3 levels of suction force (Factory setting)
- (2) Stimulation and extraction modes can be switched between by pressing the Stimulation/Expression phase button.

TROUBLESHOOTING

If there is low or no suction

- Check all connections if they are all properly attached at all connection points.
- Check the valve membrane if it is seated securely and flat on the valve body.
- Check whether the cap plug is seated properly.
- Make sure the tubing isn't kinked and blocked. Clean the tubing if there is something small inside, and dry the tubing before using it.
- Make sure all plastic parts are not distorted by high temperature sterilization.

The pump can not switch on

- Make sure the adapter is well connected to the wall socket and the pump unit.
- Make sure the polarity of the AA Alkaline batteries is correctly positioned.

Milk backflow

- If milk or water is found in the bladder or tubing, please rinse the tubing with clean hot water. Hang to AIR DRY COMPLETELY BEFORE USING. If bladder damaged, please replace a new one.

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